

## **TECHNICAL DATA SHEET**

## CANNULA TEST

PRODUCT CODE	:	4AC	
RELATED STANDARDS	:	4A Medical Cannula Test is under the scope of ISO 13485 AND 9001 standards.	
INTENDED USE	:	Cannula test is used to control whether cleaning process is successful or not.	
OPERATION CONDITIONS	:	Surgical devices are placed into the wire basket before washing Cannula Test is used with device called cannula. Cannula Test is from the aluminium foil. It is placed into the cannula device and wire basket. It is sent to the washing process. If there is any resid washer test after washing process, it means that washing process successful and it should be repeated. If there is no residue it mean process is successful.	
		Before Washing	After Washing
STORAGE AND DISPOSAL	:	Each chemical cannula test is disposable damaged products. Keep products at roc contains 100 pieces.	