

ш

Ш Т

S

⋖

C E

## **TECHNICAL DATA SHEET**

## **WASHER TEST**

PRODUCT CODE : 4AWT

**RELATED STANDARDS** : 4A Medical Washer Test is under the scope of ISO 13485 AND 9001

standards.

INTENDED USE : Washer test is used to control whether cleaning process is successful or not.

**OPERATION CONDITIONS**: Surgical devices are placed into the wire basket before washing process.

Washer test is removed from the aluminium foil. It is placed into the purple case. Purple case purpose is challeging the washer. It is sent to the washing process. If there is any residue on the washer test after washing process, it means that washing process is not successful and it should be repeated. If

there is no residue it means washing process is successful.

**PRODUCT** Each chemical washer test is disposable. (single use only). Do not use

**SPECIFICATIONS**: damaged products. Keep products at room temperature 25°C. Each box

contains 100 pieces.

PACKAGING : 50 pcs in pack / 300 pcs in a cardbox.