

TECHNICAL DATA SHEET

HOLLOW WASHER TEST

TECHNICAL DATA SHEET	PRODUCT CODE	: 4AHLWWT
	RELATED STANDARDS	: The Hallow Washer Test is under the scope of ISO 13485 AND 9001 standards.
	INTENDED USE	: Hallow Washer Test is used to control whether cleaning process is successful or not.
	OPERATION CONDITIONS	: Surgical devices are placed into the wire basket before washing process. Hollow Washer Test is used with device called cannula. Cannula Test is removed from the aluminium foil. It is placed into the Hollow Wahser device. It is sent to the washing process. If there is any residue on the washer test after washing process, it means that washing process is not successful and it should be repeated. If there is no residue it means washing process is successful.
	PRODUCT SPECIFICATIONS	: Each box contains one tube and 25 pieces canula indicator.
	STORAGE AND DISPOSAL	: Each chemical cannula test is disposable (single use only). Do not use damaged products. Keep products at room temperature 25°C.