

TECHNICAL DATA SHEET

WASHER TEST WITH GLASS

	PRODUCT CODE	:	4AWTG
	RELATED STANDARDS	:	4A Medical Washer Test is under the scope of ISO 13485 AND 9001 standards.
НШ	INTENDED USE	:	Washer test is used to control whether cleaning process is successful or not.
DATA SH	OPERATION CONDITIONS	:	Surgical devices are placed into the wire basket before washing process. Washer test has own plastic frame, so no need to use any other frame or boxes. Washer test is removed from the aluminium foil. It is placed into the wire basket. It is sent to the washing process. If there is any residue on the washer test after washing process, it means that washing process is not successful and it should be repeated. If there is no residue it means washing process is successful.
TECHNICAL	PRODUCT SPECIFICATIONS PACKAGING	:	Each chemical washer test is disposable. (single use only). Do not use damaged products. Keep products at room temperature 25°C. Each box contains 100 pieces. 500 pcs in a cardbox.