

TECHNICAL DATA SHEET (TDS)

Product Name: Med-Cover® Barrier Cream 100gr

Category: Medical Device

Class: Class 1- other
(Non-sterile and without measurement function)



1. General Product Description

Med-cover Barrier Cream helps to protect the intact skin of the perineal area from dermatitis which is caused by incontinence of patients with limited mobility, bedridden and special needs such as intensive care patients.

2. Instructions for Use/Warning

Wear gloves before use. Apply to the area you want to create a barrier layer. Avoid contact of the open wounds. Clean the previous barrier layer before each use. Do not store the product in a refrigerator. Store at room temperature.

- First, the area to be applied should be ensured that is clean and dry.
- It should be applied at least 2 times a day in the form of a thin layer.
- In each new application, the area to be applied should be cleaned and the previous applied cream should be removed from the skin.
- Urine leakage and stool leakage in the area of incontinence should be applied in every diaper change.

3. Presentations:

Med Cover Barrier Cream product is presented as a 100 gr in a tube.
A detailed product list is given in the table below.

Barcode	Reference no	Product Name	Amount	Classification/Rule
8681020477809	MCB0010	Med-cover Barrier Cream	100 gr	Class I – other / Rule 1

4. Ingredient

Aqua, Zinc Oxide, Butyrospermum Parkii (Shea) Butter, Beeswax, Dimethicone, Prunus Amygdalus Dulcis Oil, Olea Europaea Fruit Oil, Melaleuca Alternifolia Leaf Oil, Isopropyl Palmitate, Mineral Oil, Glycerin, Panthenol (Provitamine B5), Tocopheryl Acetate (Vitamine E), Aloe Barbadensis Leaf Extract, Chamomille Recutita Flower Extract, Glyceryl Oleate, Polyglyceryl-2 Dipolyhydroxystearate, Phenoxyethanol, Ethylhexylglycerin, Sodium PCA, Sodium Lactate, Arginine, Aspartic Acid, PCA, Alanine, Serine, Valine, Proline, Threonine, Isoleucine, Histidine, Phenylalanine, Creatine, Parfum.

- 5. **General Appearance:** White, Smooth, Viscous Cream
- 6. **Ph -value:** 5,00 – 6,00
- 7. **Shelf life:** 2 years
- 8. **Primary packaging:** Laminated Tube

Packaging Details	Pack/Box/Pallet Amount	Weight	Dimensions
Pack	100gr/Pack	0,13 KGS	4,5cm x 4,5cm x 15 cm
Outer Box	118 Pieces/Box	16,5 KGS	39cm x 39 cm x 30 cm
Pallet	36 Boxes/Pallet	610 KGS	80cm x 120cm x 195cm